



Shower Chairs / Stools

Assembly

RM Bathroom products come ready assembled or may require minimal assembly.

Adjustment of the legs (if applicable)


Step 1: Using your thumb, push down the Push-Pin and slide the leg up or down until the Push-Pin clicks back into the location hole.

(In some cases there maybe a push-pin on either side of the leg.)

Step 2: Ensure the Pin/s is completely through the hole, and test to ensure the two pieces do not come apart.

Step 3: Ensure all legs are adjusted to the same height.

RM408-65 Bariatric Shower Stool – to fix the backrest please attach in the same manner ensuring the Push-Pins are correctly located.

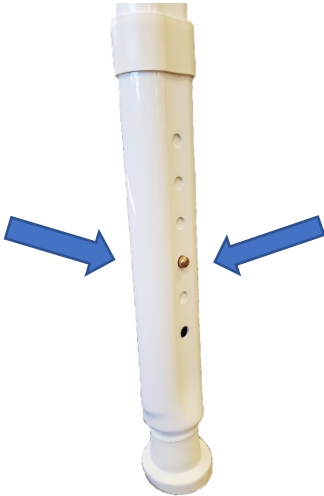

Maximum User Weight

<u>Product Code</u>	<u>Description</u>	<u>Weight Limit</u>
RM402	Shower Chair	160kg
RM403	Shower Stool	160kg
RM408-65	Bariatric Shower Stool (<i>optional backrest</i>)	305kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth. Avoid using lime scale remover. Abrasive cleaners and/or abrasive cleaning pads will seriously damage your product beyond repair.

Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance. In the event of any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impact your safety.

It is recommended that the chair is cleaned and dried off daily. This will also help to reduce the build up of limescale on the product.

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.



Imported by

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: gld@rehabandmobility.com.au

www.rehabandmobility.com.au

Your Local Dealer: