



## The Reflex Cushion


# The Reflex



The self-adjusting cushion of choice for wheelchair users with risk of tissue breakdown

### Reflex - performs after extended use

The Reflex shows consistent pressure redistribution after rigorous life-cycle\* testing. Comparison pressure testing shows that the Reflex performs as well after extended use as it does when new.

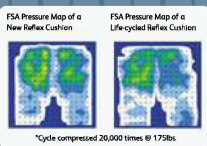

#### Available With the Following Options:

Cover: **Incontinence**

Valve: **Self-Setting Valve**

Size Range: **12x12" - 20x20"**

### Tissue Integrity

Immersion	✓✓
Dispersion away from bony areas	✓✓✓
Air/water vapour circulation	✓
Temperature balance	✓✓
Fail safe	✓✓✓
Stretch covers	Yes
Choice of covers	No
Transfer damage risk (from friction and/or shear)	Reduced

### Posture

Neutral pelvis	✓
Pre-ischial Ridge	✓✓
Stability	✓✓
Lateral support	✓✓
Thigh abduction	✓✓✓
Trochanteral ledge	✓✓
Gluteal support	✓

### Functionality

Lightweight	✓✓✓
Comfort	✓✓✓
Enables day to day activities	✓✓
Vibration dampening	✓✓✓
Low maintenance	✓✓✓
Machine washable cover/wipe down cushion	✓✓✓
Visible directions for use	✓
Functional life	✓
Warranty	2 year - cushion 1 year - cover
Size range	12x12" - 20x20"
User weight limit	None
Usability	✓✓✓
Stability	✓✓

Your Local Agent

QLD 6-8 Imboon St, Deception Bay 4508  
 Ph 1300 368 085    qld@rehabandmobility.com.au  
 www.rehabandmobility.com.au

## The Reflex Cushion



### 1. Air-Foam Floatation

The Reflex foam is perforated in the sacral-ischial area for maximum protection of the pressure sensitive area of the ITs. The Reflex provides medium density foam for the thigh trough, pelvic bucketing and pre-ischial bar, and firm foam to provide lateral support and a medial thigh separator.

### 2. Fabric

Reflex foam is bonded to puncture and water resistant polyester fabric for strength and durability.

### 3. Self-Setting Valve

A built-in air-release device automatically releases a portion of air when the user sits on the cushion. The air-release device allows a fixed amount of immersion and limits bottoming out. When weight is taken off the cushion, it automatically reinflates.

### 4. Cover

The Reflex features a tailored, removable, incontinence cover. The underside is a rugged nonslip material with hook and loop for added security. The cover is machine washable.



Reflex	12x12" (30x30cm)	12x14" (30x35cm)	14x14" (35x35cm)	14x16" (35x40cm)	14x18" (35x45cm)	14x20" (35x50cm)	15x15" (38x38cm)	15x17" (40x35cm)	16x14" (40x35cm)	16x16" (40x40cm)
w/ Incontinence Cover	73522	73524	73544	73546	73548	73540	73555	73557	73564	73566
	16x18" (40x45cm)	16x20" (40x50cm)	17x17" (43x43cm)	18x16" (45x40cm)	18x18" (45x45cm)	18x20" (45x50cm)	20x16" (50x40cm)	20x18" (50x45cm)	20x20" (50x50cm)	
w/ Incontinence Cover	73568	73560	73577	73586	73588	73580	73506	73508	73500	

Reflex Spare Covers	12x12" (30x30cm)	12x14" (30x35cm)	14x14" (35x35cm)	14x16" (35x40cm)	14x18" (35x45cm)	14x20" (35x50cm)	15x15" (38x38cm)	15x17" (40x35cm)	16x14" (40x35cm)	16x16" (40x40cm)
Incontinence Cover	05534	05535	05536	05537	05538	05539	05540	05541	05542	05543
	16x18" (40x45cm)	16x20" (40x50cm)	17x17" (43x43cm)	18x16" (45x40cm)	18x18" (45x45cm)	18x20" (45x50cm)	20x16" (50x40cm)	20x18" (50x45cm)	20x20" (50x50cm)	
Incontinence Cover	05544	05545	05546	05547	05548	05549	05578	05550	05551	

**Your Local Agent**

**QLD 6-8 Imboon St, Deception Bay 4508**  
**Ph 1300 368 085** [qld@rehabandmobility.com.au](mailto:qld@rehabandmobility.com.au)  
[www.rehabandmobility.com.au](http://www.rehabandmobility.com.au)