



RM144-46
RM144-51

G4 Aluminium Wheelchair

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.



Imported by

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: gld@rehabandmobility.com.au

www.rehabandmobility.com.au

Your Local Dealer:

Assembly / Adjustment

Congratulations on the purchase of your new G4 Aluminium Wheelchair. Equipped with several features and accessory choices, you are sure to enjoy the benefits of this chair.

Please follow these simple steps to assemble your wheelchair –

1. With the chair upright, push down on the sides of the seat until it clamps into position.
2. Height adjustable canes can be adjusted for the individual career by loosening the screws at the base of the cane, adjust and retightening.
3. Backrest is tension adjustable to suit each individual, these are adjusted by removing the cover and adjusting the velcro tension straps. Replace the cover and secure the velcro.
4. The G4 also offers a recline backrest, this is adjusted by pulling the strap at the rear where the backrest and seat meet, ensure it clicks securely into place at the desired angle.
5. Fit legrests (both left and right) into the fitting from an outward angle and turning inward until it locks into position. Footplate height can be adjusted using the allen key provided. Loosen the footplate and select the desired height, retighten to secure. Elevating legrest are available for this chair and fit into place the same way as the standard legrests. To adjust loosen the screw, select the desired angle and retighten.
6. G4 Self Propel Wheelchairs have a quick release 24" wheel. To release push the centre button on the axle and pull out. To reinstate locate the axle back into bracket on the frame. Ensure it is fully engaged before using the chair.
7. Armrests flip back and are removeable, height and depth adjustable. To release the armrest push the clip at the front and flip back, to reinstate clip back into the front bracket and ensure it locks into place. To remove armrest push clip at the front and pull lever at the rear then lift armrest out. Again ensure clipped back in securely when replacing. To adjust height and depth of armrest pad press the relevant release button and lift or slide to desired position.
8. Care-assist hand brake, (where fitted) provides added security for the person pushing the chair. When the wheelchair is stationary, the hand brake at the wheel should be applied for added security. To apply wheel brake push forward, to release pull back.

9. Anti-tip wheels can also be fitted to this chair, they provide safety when the user is ascending ramps and steep inclines, preventing the wheelchair from tipping backward. If not required they can be adusted up.
10. To fold your Wheelchair, ensure both wheel brakes are locked. Flip the footplatess up and remove. Lift the seat in the middle (holding front and back) and pull up. The chair sides will come together. At this time you can also remove the large Self Propel wheel if required to make the chair lighter and more compact for stowing.

Maximum User Weight

Product Code	Description	Weight Limit
RM144-46	G4 Wheelchair Self Propel 46cm seat	150kg
RM144-46D	G4 Wheelchair S/P 46cm seat with Drum Brake	150kg
RM144-51	G4 Wheelchair Self Propel 51cm seat	150kg
RM144-51D	G4 Wheelchair S/P 51cm seat with Drum Brake	150kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you carry out regular checks: all nuts, bolts, knobs and handles should remain secure. Wheels should move freely and brakes engage when applied. To ensure the best from your rollator it should be kept well maintained.

In the event of identifying any irregularities you should contact your local distributor for advice.

NB: Your wheelchair should only be used for its intended purpose.