



RM160-56

RM160-61

## G6 Excel Bariatric Wheelchair

# Warranty

---

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.

---



## **Imported by**

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: [gld@rehabandmobility.com.au](mailto:gld@rehabandmobility.com.au)

[www.rehabandmobility.com.au](http://www.rehabandmobility.com.au)

**Your Local Dealer:**

# Assembly / Adjustment

Congratulations on the purchase of your new G6 Excel Wheelchair. Equipped with several features and accessory choices you are sure to enjoy the benefits of this chair.

Please follow these simple steps to assemble your wheelchair –

1. With the chair upright, push down on the sides of the seat until it clamps into position.
2. Height adjustable canes can be adjusted for the individual career by loosening the screws at the base of the cane, adjust and retightening.
3. Fit legrests (both left and right) into the fitting from an outward angle and turning inward until it locks into position. Footplate height can be adjusted using the allen key provided. Loosen the footplate and select the desired height, retighten to secure. Elevating legrest are available for this chair and fit into place the same way as the standard legrests. To adjust loosen the screw, select the desired angle and retighten.
4. G6 Self Propel Wheelchairs have a quick release 24" wheel. To release push the centre button on the axle and pull out. To reinstate locate the axle back into bracket on the frame. Ensure it is fully engaged before using the chair.
5. Armrests flip back and are removeable. To release the armrest push the clip at the front and flip back, to re-instate clip back into the front bracket and ensure it locks into place. To remove armrest push clip at the front and pull lever at the rear then lift armrest out. Again ensure clipped back in securely when replacing.
6. Care-assist hand brake provides added security for the person pushing the chair. When the wheelchair is stationary, the hand brake at the wheel should be applied for added security. To apply wheel brake push forward, to release pull back.

7. Anti-tip wheels can also be fitted to this chair, they provide safety when the user is ascending ramps and steep inclines, preventing the wheelchair from tipping backward. If not required they can be adjusted up.
8. To fold your Wheelchair, ensure both wheel brakes are locked. Flip the footplates up and remove. Lift the seat in the middle (holding front and back) and pull up. The chair sides will come together. At this time you can also remove the large Self Propel wheel if required to make the chair lighter and more compact for stowing.

---

## **Maximum User Weight**

<b>Product Code</b>	<b>Description</b>	<b>Weight Limit</b>
RM160-56	G6 Wheelchair Self Propel 56cm seat	250kg
RM160-56D	G6 Wheelchair S/P 56cm seat with Drum Brake	250kg
RM160-61	G6 Wheelchair Self Propel 61cm seat	250kg
RM160-61D	G6 Wheelchair S/P 61cm seat with Drum Brake	250kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

---

## **Care & Maintenance**

All of the product components are manufactured from high quality materials. However, it is very important that you carry out regular checks: all nuts, bolts, knobs and handles should remain secure. Wheels should move freely and brakes engage when applied. To ensure the best from your rollator it should be kept well maintained.

**In the event of identifying any irregularities you should contact your local distributor for advice.**

**NB: Your wheelchair should only be used for its intended purpose.**