



RM836-SP Self Propel  
RM836-AP Attendant Propel

---

## Compact Lite Wheelchair

---

# Assembly / Adjustment

Congratulations on the purchase of your new Compact Lite Aluminium Wheelchair. Equipped with several features and accessory choices you are sure to enjoy the benefits of this chair.

Please follow these simple steps to assemble your wheelchair –



1. With the chair upright, pull each legrest out and down into 'C' cup locator. Footplate height can be adjusted using the allen key provided.
2. Push down on the sides of the seat until it clamps into position.
3. Lift backrest up to click into position. Depress black latch to fold.
4. Armrests flip back to enable closer position to table / desk.
5. Care-assist hand brake, provides added security to the person pushing the chair. When the wheelchair is stationary, the hand brake at the wheel should be applied for added security. To apply push forward, to release pull back.
6. Anti-tip wheels can also be fitted to this chair, they provide safety when the user is ascending ramps and steep inclines, preventing the wheelchair from tipping backward. If not required they can be adjusted up.

7. To fold your Wheelchair, ensure both brakes are locked. Flip the footplates up and lift legrest up next to seat. Lift the seat in the middle (holding front and back) and pull up. The chair sides will come together.
8. Lift wheelchair by the frame, not the armrests.

---

## **Maximum User Weight**

<b>Product Code</b>	<b>Description</b>	<b>Weight Limit</b>
RM836-SP	Compact Lite Self Propel Wheelchair	110kg
RM836-AP	Compact Lite Attendant Propel Wheelchair	110kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

---

## **Care & Maintenance**

All of the product components are manufactured from high quality materials. However, it is very important that you carry out regular checks: all nuts, bolts, knobs and handles should remain secure. Wheels should move freely and brakes engage when applied. To ensure the best from your rollator it should be kept well maintained.

**In the event of identifying any irregularities you should contact your local distributor for advice.**

**NB: Your wheelchair should only be used for its intended purpose.**

# Warranty

---

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.

---



## **Imported by**

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: [gld@rehabandmobility.com.au](mailto:gld@rehabandmobility.com.au)

[www.rehabandmobility.com.au](http://www.rehabandmobility.com.au)

**Your Local Dealer:**