



RM440


Adjustable Toilet Seat Raiser

Installation



Step 1 Remove existing toilet seat from the bowl.

Step 2 Position the unit onto your toilet bowl, inserting the rear mounting bolts through the existing holes on your toilet.

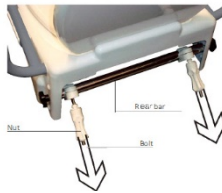

Step 3 The position of the mounting bolts can be adjusted to match different toilets. To adjust, loosen & slide the mounting bolts along the rear bar to achieve the correct distance. Reattach nuts and tighten firmly. *Ensure toilet seat raiser is firmly clamped against the bowl before use.*

Adjusting seat height


The toilet seat raiser can be adjusted to 3 different height settings, adding 50, 100 or 150mm to the height of the bowl. Height can be adjusted either before or after installation.

Flip up the toilet seat for easy access to the bars.

To adjust the height, undo the front thumb screw bolts, reposition the bar to the desired height and reinsert the thumb screw bolts on both sides and tighten until firm.

Adjust the rear bar in the same way. Adjustment can be made to facilitate a forward angled tilt for users requiring extra assistance when standing (rear bar lower than the front bar).


When adjustment is complete, replace all thumb screw bolt protector caps if the unit is supplied with alternative bolt and cap combinations.

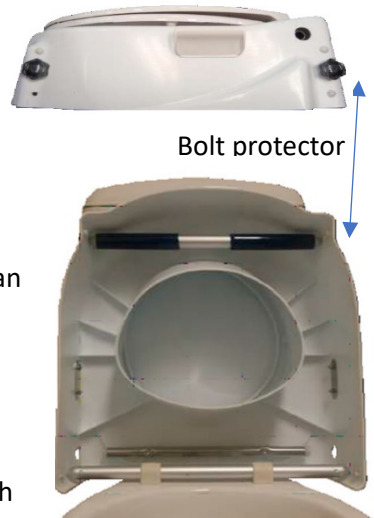



Ideal Seat Height

Ideal height is when the user's feet are flat on the floor, hip height is higher than knee height (ie less than 90 degrees on the knee joint or hip joint)

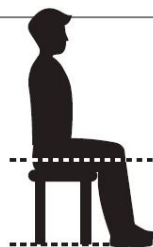

Inserting Armrests

The toilet seat raiser can be used with or without armrests. One or both armrests can be used to suit individual user. A brass locking button is used to secure each arm.

Insert the armrest by pressing the locking button located under the frame of the armrest and slide the armrest into the arm hole at the side of the toilet seat. Ensure secure before use.

NB: The toilet seat raiser operates just like a standard toilet seat. The seat may be flipped up so the bowl can be accessed without the seat.

Maximum User Weight

Product Code	Description	Weight Limit
RM440	Toilet Seat Raiser	136kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you only clean your toilet seat raiser using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners and/or abrasive cleaning pads will seriously damage your product beyond repair.

Regularly check the mounting screws are tight and that the toilet seat raiser is firmly clamped to the toilet bowl. *Do Not use if the seat is cracked or deformed in any way.*

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.



Imported by

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: gld@rehabandmobility.com.au

www.rehabandmobility.com.au

Your Local Dealer: