



Adjustable Bed Cradle



VY440

Assembly and Operating Instructions

INTRODUCTION

Thank you for deciding to purchase the Aidapt Adjustable Bed Cradle. The Adjustable Bed Cradle is manufactured from the finest material and components available, when installed and used correctly it is designed to give many years of reliable trouble-free service. The Adjustable Bed Cradle can be fitted to single, double, queen and king size beds.

The Height Adjustable Bed Cradle from Aidapt is a metal frame designed to keep bed covers away from the body. It is useful for those suffering from ulcers or skin irritation.

N.B. This equipment must be installed by a competent person and a risk assessment undertaken as to the suitability of the bed cradle for the particular user and user's bed.


INSTALLATION

The base is worked between the mattress and bed base and can be used at the side or end of the bed.

HEIGHT ADJUSTMENT

The height of the Adjustable Bed Cradle can be adjusted to suit the client's requirements. To adjust the Adjustable Bed Cradle simply remove the two grey e-clips and lift to the required height ensuring that you do not exceed the last pre-punched hole on the base of the frame.

Once the desired height is achieved, re-insert the e-clips into the pre-punched holes, ensuring that both clips pass completely through the tube frame, the inner tube and out the other side and are in place securely as shown below.

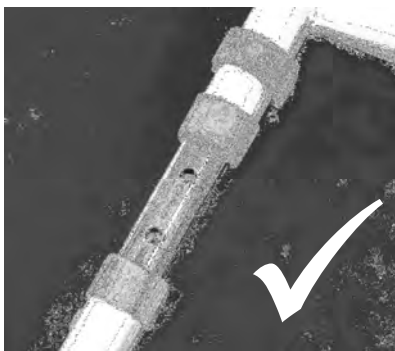

CORRECT



INCORRECT

IMPORTANT!

The Adjustable Bed Cradle is not intended as an aid to help the user in and out of bed.


NB. Do not lean on the Bed Cradle or apply heavy loads.

The Bed Cradle has a maximum load of 5 kg and is designed to hold bed clothes only.

CLEANING

The Adjustable Bed Cradle should be cleaned using a non-abrasive cleaner or mild detergent with a soft cloth only. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

TECHNICAL SPECIFICATION

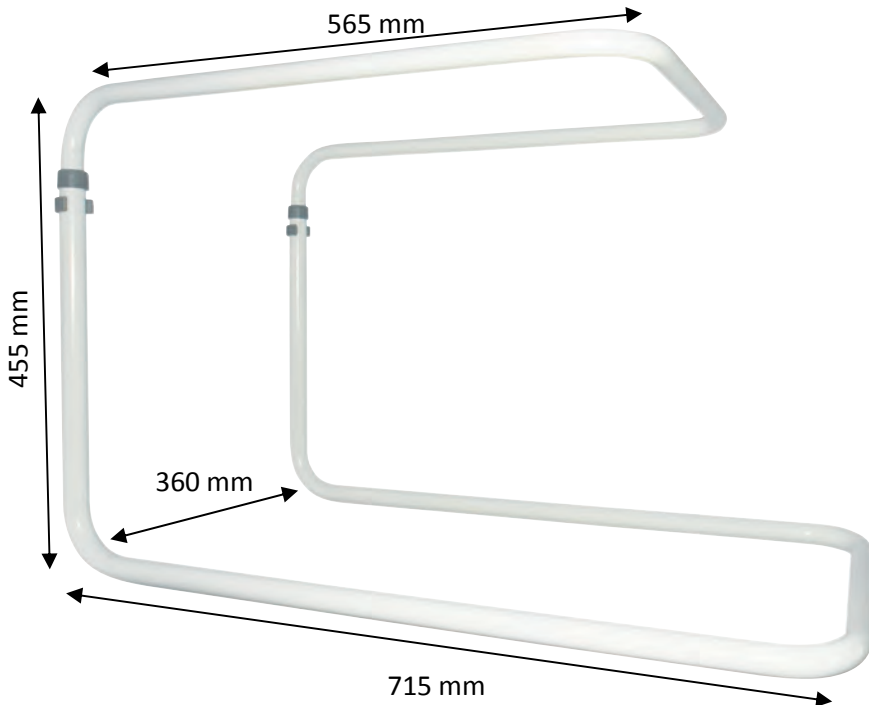

MAINTENANCE

Please check the cradle periodically for any signs of damage and if in doubt contact your supplier.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors and other screw in/bolt in/push in components.
- Upholstery (if applicable) for security, splits, exposed staples, etc.
- Straightness and integrity of all tubing and welds.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment must be undertaken as to the suitability of the Bed Transfer Aid for the particular user and user's bed.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Rehab and Mobility Wholesalers
6-8 Imboon Street, Deception Bay, QLD 4508
qld@rehabandmobility.com.au
1300 368 085