



Broadstairs Toilet Frame

VR202 (Free-Standing)

VR203 (Floor Fixed Version)



Fixing and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

INTRODUCTION

Thank you for deciding to purchase the Broadstairs Toilet Frame from Aidapt. The Broadstairs Toilet Frame is manufactured from the finest materials and components available, when used correctly it is designed to give many years of reliable trouble-free service.

Please Note: Always consult with the prescriber/Physiotherapist if you are in any doubt that this product is suitable for your use.

NB. This equipment must be installed by a competent person.

190 kg (30 st) weight limit.

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage, if you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your toilet frame using a non-abrasive cleaner or mild detergent with a soft cloth and avoid using lime scale remover.

Under no circumstances use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.

NB. Regular daily cleaning will reduce the build-up of limescale on your product.

Body spillages should be cleaned as and when they occur (giving particular attention to the area where the frame width is extended), as this will avoid causing rust or corrosion which could damage/weaken the frame.




Our equipment can be cleaned and or disinfected as per MHRA guidance on decontamination MHRA DB 2003 (06). This product is suitable for cleaning with moist heat or autoclaving. When using moist heat the equipment can be held at 81°C for at least one minute to achieve disinfection.

ADJUSTMENTS TO HEIGHT EXTENSION LEGS

- Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (See fig. 1)
- Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (See fig. 2).
- Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

ADJUSTMENTS TO WIDTH EXTENSION BARS

- Remove the 'E' clip from the extension bar and extend the frame to the desired width (See fig. 1)
- Once you have achieved the desired width, reinstall the 'E' clip, ensuring that it has passed through the extension bar and is fitting neatly and snug around the outside leg. (See fig. 2).
- Ensure the e-clip has passed through BOTH the external tube and the internal extension bar (See fig. 3) Do not use if the e-clip is only through the internal extension bar (See fig. 4)

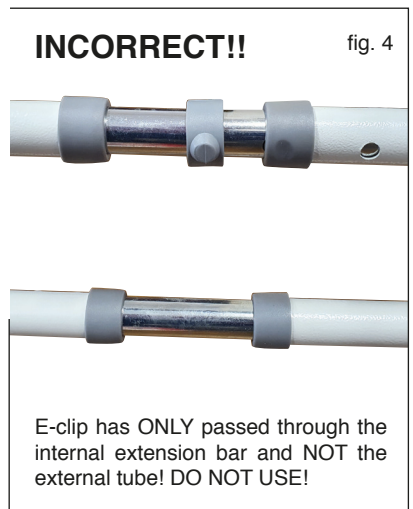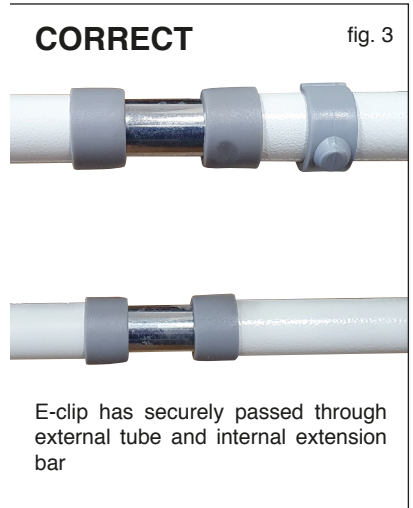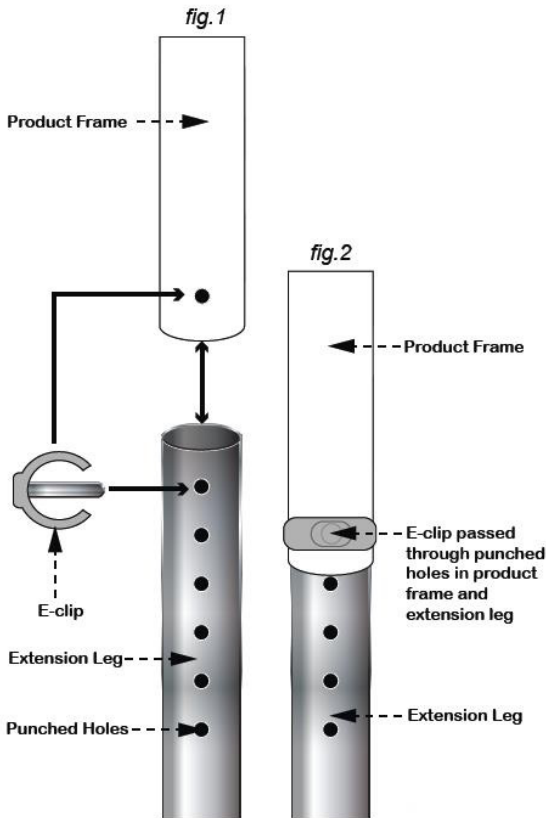

FITTING OF THE FLOOR FIXING KIT (OPTIONAL EXTRA SUPPLIED WHEN ORDERING VR203)

Remove the 'E' clips from the extension legs and completely remove the extension legs from the frame. Insert the supplied floor fixing legs. Select the desired height setting and insert the 'E' clips to fix the legs in place. Position the frame into the desired place around the toilet and use appropriate screws (not supplied) to secure the fixing plates to the floor.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Proper height adjustment
- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.
- Also check all upholstery for security, splits, etc.
- Width and leg extensions

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns. Please pay particular attention to the width and leg extensions.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Rehab and Mobility Wholesalers
6-8 Imboon Street, Deception Bay, QLD 4508
qld@rehabandmobility.com.au
1300 368 085**