



Commodes & Toilet Frames

Fixing and Maintenance Instructions

Product Code	Description	Weight Limit
VR157	Solo Skandia Toilet Seat and Frame	154 kg (25 st.)
VR157B	Solo Skandia Bariatric Toilet Seat and Frame	254 kg (40 st.)
VR158	Solo Skandia Toilet Seat and Frame	154 kg (25 st.)
VR158B	Solo Skandia Bariatric Toilet Seat and Frame	254 kg (40 st.)
VR160	Kent Commode (Pre-assembled)	170 kg (27 st.)
VR160G	Kent Commode (Flat packed)	158 kg (25 st.)
VR161	Essex Commode (Brown)	170 kg (27 st.)
VR161BL	Essex Commode (Blue)	170 kg (27 st.)
VR161BLA	Essex Commode (Light Blue)	170 kg (27 st.)
VR161G	Essex Commode (Grey)	154 kg (25 st.)
VR162	Surrey Commode	170 kg (27 st.)
VR213	Ashby Lux Toilet Seat and Frame	190 kg (30 st.)
VR215	Solo Skandia Toilet Seat and Frame with Lid	154 kg (25 st.)
VR219	President Toilet Seat and Frame	154 kg (25 st.)
VR219B	President Bariatric Toilet Seat & Frame	254 kg (40 st.)
VR220	President Toilet Seat and Frame	154 kg (25 st.)
VR221W	Sussex Bariatric Commode	254 kg (40 st.)
VR224	Cosby Bariatric Toilet Seat and Frame	254 kg (40 st.)
VR226W	Dorset Bariatric Commode	254 kg (40 st.)
VR227W	Devon Bariatric Commode	254 kg (40 st.)
VR228W	Suffolk Bariatric Commode	254 kg (40 st.)
VR233	Ambassador Raised Toilet Seat	154 kg (25 st.)
VR235	Norfolk Commode	170 kg (27 st.)
VR240	Solo Skandia Toilet Seat and Frame with Lid	154 kg (25 st.)
VR264	Ashby Commode	160 kg (26 st.)
VR276	Solo Skandia Economy Toilet Seat and Frame with Lid	127 kg (20 st.)

Do not exceed the weight limit stated – doing so could put the user at risk.
NB. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the product for the particular user.

READ BEFORE USE

Before using your new product, you and each person that may assist you in its use should take time to read this manual and follow all instructions.


INTRODUCTION

Thank you for deciding to purchase your Commode or Toilet Seat from Aidapt. This product is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

ALL COMMODES

Please examine your commode for any visible damage before using. If you see any damage or suspect a fault, please contact your supplier.

In this case, please do not use your product as it may compromise your safety.

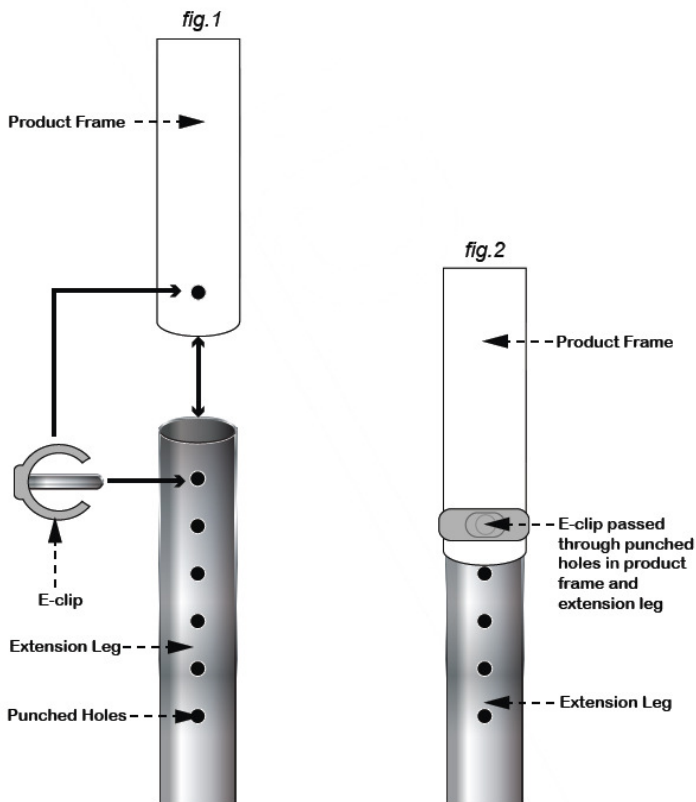

ADJUSTMENT OF LEGS (IF APPLICABLE)

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the height you need, re-install the 'E' clip ensuring that it has passed through the extension leg and is fitting neatly and snugly around the commode leg. (see fig. 2)

Make sure that all legs are sitting evenly onto the floor; never extend legs in a manner that the commode is slanting. This will compromise the safety of the user.

SURREY/SUFFOLK COMMODOES ONLY

The Surrey/Suffolk Commode has detachable arms; to remove these simply lift the arms out the locating tubes welded to the side of the commode frame. Always ensure that the arms are re-fitted neatly and evenly into the locating tubes before use. Failure to do so will compromise the safety of the user.

COMMODOE BUCKETS (ALL COMMODOES)

To remove the commode bucket, simply lift the toilet seat and fix the lid provided onto the bucket to avoid splashing and carefully raise the bucket and dispose of the contents into a proper sluice or WC. Never use an open drain, sink or basin. This could cause a serious health risk. Clean out the residue with a suitable disinfectant and refit to the commode after use

CLEANING (ALL COMMODOES)

All commodes must be cleaned using a non-abrasive cleaner or can be pressure washed. However, always ensure that any chemicals you use with your pressure washing will not affect chrome plate, polymer coating or epoxy powder coating. After cleaning use a water dispersing agent such as WD40 on all joints and castors to reduce the chance of premature rust or seizure.

WHEELS (IF APPLICABLE)

The two small wheels attached are to assist in moving and positioning the commode. They must not be used to move the commode whilst in use or when somebody is seated on the commode.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- PAT test for electrical items
- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.
- Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk