

Peak Deluxe Folding Walker

Product Codes: 9346376060400, 9346376060417

Thank you for purchasing the Peak Deluxe Folding Aluminium Walker.

Peak Deluxe Folding Aluminium Walkers are designed to assist people who have difficulty in walking.

Please take time to go through this manual which gives you more details about your Walker. Please keep this booklet handy and use it as and when required. Peak Deluxe Folding Aluminium Walker is warranted to be free of defects in material and workmanship for twelve months from the date of purchase. This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers. In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as wheels and rubber tips which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor.

Peak Care Equipment
149 Princes Highway
Unanderra NSW 2526
Tel: +612 4272 2688
www.peak-care.com.au

Distributor:
Rehab and Mobility Wholesalers
6-8 Imboon Street, Deception Bay,
QLD 4508
1300 368 085
qld@rehabandmobility.com.au



User Manual

Features

- Light Weight
- Easy to fold with single push-button
- Interchangeable legs from tips to wheels
- Maximum User Weight Capacity of 130 Kg (290lbs)

Safety Instructions

- A risk assessment should always be carried out to ensure the product is suitable.
- Ensure all components are properly and securely installed.
- If normal function of the castors is not observed, return the Walker to the place of purchase for adjustment or replacement of parts.
- Ensure the user has sufficient strength to safely operate and control the Walker.
- Do not exceed the maximum recommended working load.
- Avoid storing the Walker in direct sunlight; handles or parts exposed to sunlight will heat up and may cause injury to the skin.
- When using outdoors, use only on designated footpaths or walkways; do not use on roads, streets or highways.
- Do not use the Walker on any incline greater than 9 degrees, or on a slope with water, ice or snow covering.
- Use caution when traversing kerbs or obstacles as this may cause the Walker to tip over and cause bodily harm or damage to the Walker

Assembly Instruction

1. Remove all contents from carton: Frame and Legs
2. Open each side of the frame until you hear a click
3. Attach the 4 Legs

Finding the Right Height

Elbow should be at 30 Degrees when standing and holding onto the walker handles.

