



Shower Chairs



Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase Shower Stool from Aidapt. These Shower Stools are manufactured from the finest materials and components available, when used correctly they are designed to give many years of reliable trouble-free service.

IMPORTANT!

N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.

MODELS AND WEIGHT LIMITS

VB499SAL	Milton Shower Chair (Aluminium)	150 kg (23 st.)
VB499/VB500	Milton Shower Chair	127 kg (20 st.)
VB499S/VB500S	Milton Shower Chair	127 kg (20 st.)
VB501	Luton Shower Chair	159 kg (25 st.)
VB648/VB506	Hythe Shower Chair	190 kg (30 st.)
VB650/VB506A	Hythe Adjustable Shower Chair	127 kg (20 st.)
VB507	Charing Shower Chair	190 kg (30 st.)
VB510	Darenth Shower Chair	190 kg (30 st.)
VB509	Rochester Shower Stool	190 kg (30 st.)
VB508	Dartford Shower Stool	190 kg (30 st.)
VB635/VB635H	Strood Shower Stool	159 kg (25 st.)
VB636	Pembury Corner Shower Chair	159 kg (25 st.)
VB638W	Portland Bariatric Shower Chair	254 kg (40 st.)
VB639W	Portland Bariatric Shower Chair with Padded Back	254 kg (40 st.)

DO NOT exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the walking stick.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

ADJUSTMENTS TO EXTENSION LEGS

Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2).

Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

USE OF BATH OILS

Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance.

In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.


CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair.

Certain products have clip on/clip off backs and/or seats; you can easily remove these if required, clean with a mild detergent and soft cloth in a warm water solution. Do not exceed 50°C when cleaning the seat cover and/or back as this will permanently damage the plastic.

N.B. It is recommended that the chair is cleaned and dried off daily. This will also help to reduce the build up of limescale on the product.

UNDER NO CIRCUMSTANCES use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.

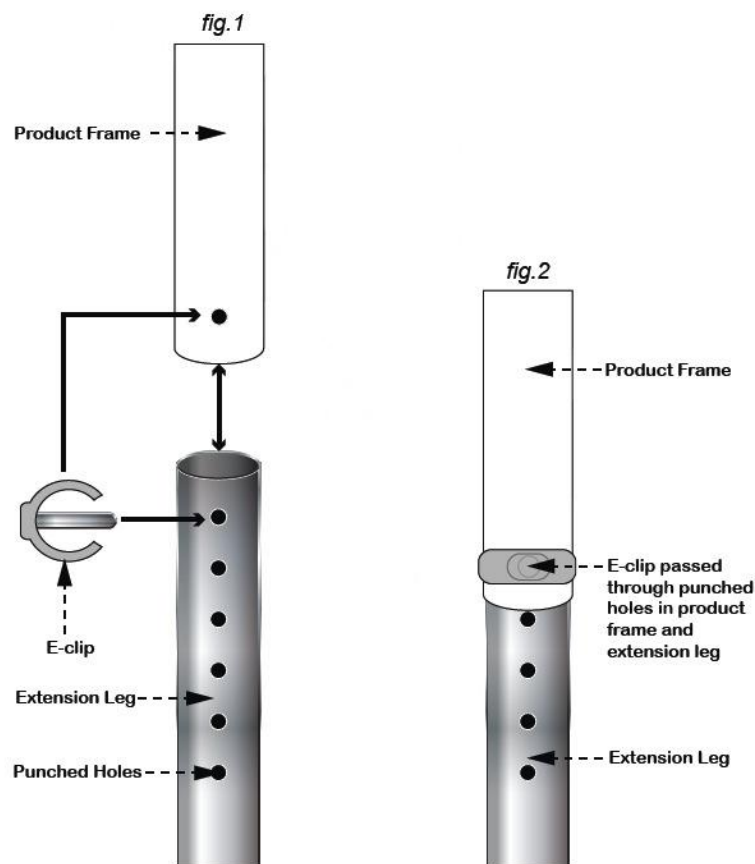

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals and on reissue, or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- PAT test for electrical items
- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.
- Also check all upholstery for security, splits, etc.
- Rust or damage to tubing

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk