




Over Toilet Aids / Commodes

Assembly

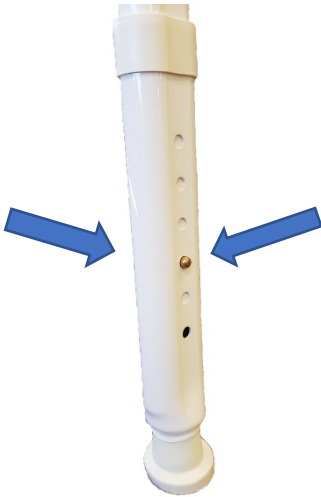

RM Bathroom products come ready assembled.

Adjustment of the legs (if applicable)

Step 1: Using your thumb, push down the Push-Pin and slide the leg up or down until the Push-Pin clicks back into the location hole.

(In some cases there maybe a push-pin on either side of the leg.)

Step 2: Ensure the Pin/s is completely through the hole, and test to ensure the two pieces do not come apart.

Step 3: Ensure all legs are adjusted to the same height.

Maximum User Weight

Product Code	Description	Weight Limit
RM400	Over Toilet Aid	160kg
RM406-65	Bariatric Over Toilet Aid	305kg
RM3327	Bariatric Static Commode	226kg
RM371	Combination Over Toilet Aid / Shower Chair	227kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you only clean your over toilet aid or commode using a non-abrasive cleaner or mild detergent with a soft cloth. Avoid using lime scale remover. Abrasive cleaners and/or abrasive cleaning pads will seriously damage your product beyond repair.

All Over Toilet Aids & Commodes should only be used for their intended purpose.

All Over Toilet Aids & Commodes should be inspected regularly to ensure moving components (push-pins, seats) remain secure. If in doubt, do not use and immediately contact your supplier for support.

Your safety is of the utmost importance.

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.



Imported by

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: gld@rehabandmobility.com.au

www.rehabandmobility.com.au

Your Local Dealer: