


ADJUSTING HANDLE BAR

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet. Take the C-cap out, draw the handle tube up and down to the desired height. As Figure 4, insert the C-cap through the hole.

USING BRAKES

1. Grasp both handles and brake bar, hold up the brake bar.
2. To lock in brake, push down on the brake bar. Lift brake bar to release.

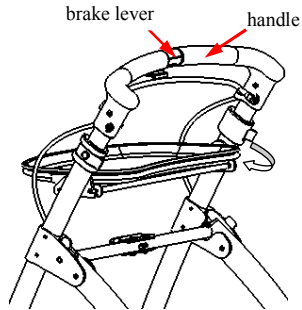

Figur5

USING TRAY

1. Ensure the tray well installed with the rollator before using. If not installed, follow the steps to re-fit it.
2. Please support prominent position when you take off the it, hold upwards the tray, as figure 6.

BAG CLEANING

Clean the bag with water or soft cleaner.

CARE AND MAINTENANCE

1. Ensure that **ALL** attaching hardware is secured at **ALL** times.
2. Check screw to secure folding lever is locked. If not. Tighten if necessary.
3. Check the clip is fully snapped in the regulating hole.
4. Replace any broken, damaged or worn items immediately.

CARE OF WHEELED ACCESSORIES

1. Clean wheels with warm water and mild cleanser. Dry with clean rag.
2. Lubricate with 3-in-1 oil periodically or if wheels begin to squeak.

9290 Indoor Rollator User Manual

NOTE: Check all parts for shipping damage. In case of shipping damage, Do NOT use. Contact Dealer/Carrier for further instruction.

SAFETY SUMMARY

To ensure the safe operation of the **GENTEEL ROLLATOR**, these instructions **MUST** be followed:

GENERAL WARNING

DO NOT install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment – otherwise, injure or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

A physical/occupational therapist should assist in the height adjustment of the rollator for maximum support and correct brake activation.

Always ensure hand rest tubes are well fixed without sway under load by C-cap well-inserted. Failure to follow these warnings can result in personal injury

Care should be taken to ensure that **ALL** hand and height adjustment are secure and that casters and moving parts are in good working order before using.

GENERAL WARNING

All wheels **MUST** be in contact with the floor at **ALL TEMES** during use. This will ensure the rollaor is properly balanced. The brakes **MUST** be in the locked position **BEFORE** dining.

When using the indoor rollator in a stationary position, the hand brakes **MUST** be locked.

STABILITY WARNING

The indoor rollator can provide ambulatory assistance to an individual weighing up to 100kg, **INCLUDING** the weight of the contents of the bag. The bag has a weight limitation of 5kg.

The tray has a weight limitation of 5kg.

UNFOLDING OR ASSEMBLING WARNING

Ensure that all castors are well contacted with floor without any sway before usage after unfolding or assembling rollator, and handle is adjusted to the suitable position without sway with C-cap well-inserted.

Ensure the stray support tubes are safely opened without movement or loose.

ASSEMBLING WARNING

ASSEMBLING WARNING

Always test to see that the rollator and attachments are properly and securely locked in place **BEFORE** using.

Product feature

Easy to take off and fix, bag attached.

Easy-to-fold and easy to storage and transport.

Handle adjustable by snap bottom for different user need

Bar brake for easy operate

Intended use

Suitable for the elderly to transport or put food inside

Assisting the elderly to walk inside

Parameter

Max. User Weight	100kg
Overall Depth	66cm
Overall Width	54cm
Overall Height	83.5~94.5cm 5 adjustments
Tray Depth	29cm
Tray Width	39cm
Tray Height	68cm
Front/ Rear Wheel Size	7"
Brake Method	Bar brake
Tray max load	5kg
Bag max load	5kg



OPENING/FOLDING THE ROLLATOR

NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this instruction sheet.

The figures are without bag for clearer show

OPENING

1. Take the rollator out of the carton and put it upright on the floor. Support the handle with one hand and catch the front edge the tray with the other hand, pull the tray to the direction as Figure 1. The tray is completely open with a clear "click".

2. Support the handle with one hand and press on the tray with the other hand to the direction of Figure2. The rollator is completely opened for use with a clear "click"

Figure1


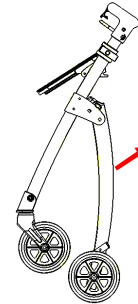


Figure2

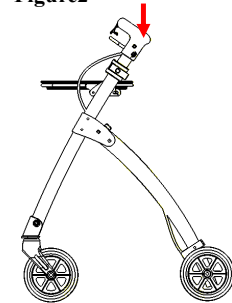

FOLDING

1. Grasp the folding bar, as figure 3, unfasten the tray.

2. Hold the handle with one hand; hold the frame folding lever with the other hand. Then hold up the folding bar and the rollator can be foldable. As figure 4.

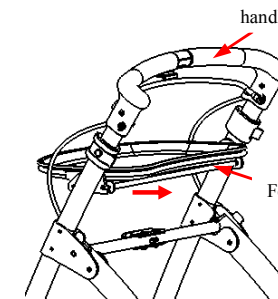


Figure 3

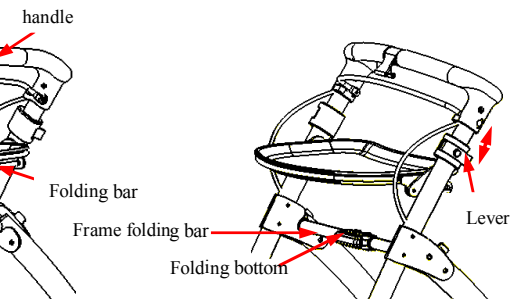


Figure 4