

Thank You!

Thank you for purchasing the Peak Raised Toilet Seat With Swing Back Arms.

Peak Raised Toilet Seat With Swing Back Arms are designed to assist with Toilet transfers, are available with or without Lids.

Kindly take time to go through this manual which gives you more details about your Toilet Seat Raiser. Kindly keep this booklet handy and use it as and when required.

Peak Raised Toilet Seat With Swing Back Arms is warranted to be free of defects in material and workmanship for twelve months from the date of purchase. This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers. In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as clips which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor.

Peak Care Equipment
149 Princes Highway
Unanderra NSW 2526
Tel: +612 4272 2688
www.peak-care.com.au

Distributor:
Rehab and Mobility Wholesalers
6-8 Imboon Street,
Deception Bay, QLD 4508
1300 368 085
qld@rehabandmobility.com.au

Peak Raised Toilet Seat With Swing Back Arms

Product Codes: C8310-A-FX



User Manual

Staple Receipt Here:

INSTALLATION

- Choose the desired increased height of product, setting both the front & rear aluminium x bars,
- ensuring all four hand wheels are tight.
- Connect both armrests, ensuring spring loaded gold button passes inside both side panels. Confirm correct position by pulling armrest to ensure it remains in place.
- Remove the existing toilet seat from toilet if applicable.
- Place the raised toilet seat with arms onto the toilet bowl, and connect the rear attachment bolts through the existing toilet bowl holes located at the rear of the bowl.

REGULAR MAINTENANCE

- For safety, precautions must be taken for the safe use of this equipment.
- The raised toilet is not designed for permanent installation. The product should be loosened and retightened once a week to ensure a proper fit.
- Regularly check products for defects. If a defect appears, do not attempt to use the product & contact the distributor immediately.
- Regularly check correct arm rest locations and front and rear aluminium x bars for tightness.
- Clean with warm soapy water or mild detergent only and wipe clean to remove residue.

USE

- Check that the product sits flat on the bowl and test for stability before each use.
- Ensure even weight distribution on both armrests when lowering & raising from toilet.
- Maximum user weight capacity: 120 kg

ABOUT

- Designed to assist users to lower onto and rise from the toilet by increasing toilet seat height.
- Increased height option: 65 mm / 105 mm / 145 mm
- Peak Care products are manufactured using the latest materials & technology & will last for many years if used in a safe and correct manner.
- The product information is supplied as a guide only. We recommend the correct usage, fitting and suitability of the product be advised by a
- QUALIFIED THERAPIST.

DISTRBUTOR

- Examine the product for possible shipping damage.
- Ensure customer has been assessed & advised by a health professional.
- Ensure customer has been briefed thoroughly about product capabilities & correct usage.