



RM804

Walking Frame

Bariatric

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.



Imported by

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: gld@rehabandmobility.com.au

www.rehabandmobility.com.au

Your Local Dealer:

Assembly / Adjustment



Step 1: Before use, ensure the walking frame is fully folded out, with both sides clicking into locked position.

Step 2: To adjust height, push leg adjustment pins inwards, extend legs to desired height and ensure pins lock back in position.

Step 3: To fold the walking frame, press both the left and the right 'easy folding lever' mechanism (*as indicated*), bringing each side in against the front frame.

NB: Before using, ensure both sides have clicked firmly into place prior to walking.

Maximum User Weight

<u>Product Code</u>	<u>Description</u>	<u>Weight Limit</u>
RM804	Bariatric Walking Frame	270kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you carry out regular checks: all nuts, bolts, knobs and handles should remain secure. Wheels (*if applicable*) should move freely. To ensure the best from your walking frame it should be kept well maintained.

In the event of identifying any irregularities you should contact your local distributor for advice.