



RM202



RM204



RM204WB-Y


RM2 Series Rollators

Assembly / Adjustment

Your new Rollator comes with the following parts:

Main frame (bag, seat attached)	1x Backrest
2x Handles	2x Handle bolts, washers & knobs

1. Place handles into the frame (1) select the desired height and tighten with bolt. Ensure the bolt is positioned on the inside of the frame and seated in the hexagonal cut-out on the frame, washer and knob tighten on the outer side of the frame.
2. Depress the push-pins on the backrest (2) and slide into place on the brackets at the top of the frame.
3. To adjust the sensitivity of the brakes, pull the brake adjustment (5) screw up to release the locking nut, then turn the locking nut to desired brake sensitivity. Clockwise to increase sensitivity, anticlockwise to reduce sensitivity.

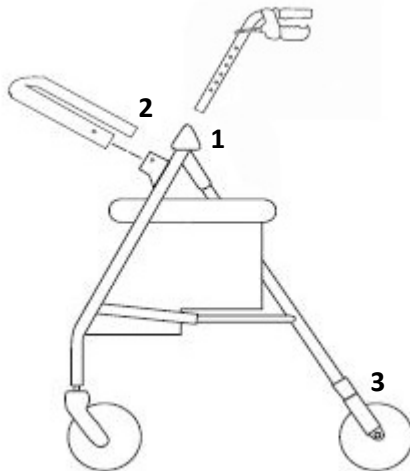

Maximum User Weight

Product Code	Description	Weight Limit
RM202	Modula Rollator	150kg
RM204	Genesis Rollator	200kg
RM204WB-Y	Genesis Wide Rollator	200kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you carry out regular checks: all nuts, bolts, knobs and handles should remain secure. Wheels should move freely and brakes engage when applied. To ensure the best from your rollator it should be kept well maintained.

In the event of identifying any irregularities you should contact your local distributor for advice.

NB: Your Rollator should only be used for its intended purpose and for highest safety you should not push anyone seated.

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.



Imported by

Rehab and Mobility Wholesalers Pty Ltd

P: 1300 368 085

E: gld@rehabandmobility.com.au

www.rehabandmobility.com.au

Your Local Dealer: