



## **Folding Ramp**



**VA141**

**Fixing and Maintenance Instructions**

## INTRODUCTION

Thank you for purchasing the Folding Ramp from Aidapt.

**These instructions should be read carefully and left with the user of the product for future reference.**

Please ensure this equipment is installed/used by a competent person.

VA141 – Roll Up Ramp **300kgs (47.2 St.) MAX Weight Load**

Do not exceed the weight limit stated – doing so could put you at risk.

## INTENDED USE

The Folding Ramp from Aidapt is a compact, folding, aluminium ramp for easy access to houses and cars for wheelchairs and scooters.

## FEATURES

- Portable, compact design that is quick to install
- Supplied with a bag
- Allows easy access to houses and cars

## ASSEMBLY INSTRUCTIONS

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

1. The ramp should be fully rolled out and the side locked and must be securely in place before use.
2. Select a place to use where the ramps have the minimum gradient possible. Ensure the gradient of the ramps is not beyond the ability of the wheelchair or occupant. The maximum gradient is 1 in 12.
3. Ensure the ramp is fully assembled and the front of the ramp is placed upon the step or obstacle being climbed (the front of the ramp is identifiable due to the cut out edges on either side).
4. Ensure the ramps are safe before use.

## TO ROLL OUT

- To begin, roll out the main section of the ramp and check to ensure it's straight.
- Attach the side sections ensuring the fastening clips are correctly locked into place (the locking pin end of the side rail should be at the same end of the holes for the locking pin).
- Check the ramp is correctly assembled.

## TO ROLL UP

- Release the latches securing the side sections to the ramp.
- Remove the side sections and roll up the main section of the ramp
- Ramp can be securely placed into provided carrier bag for easy storage.

## MAINTENANCE & SAFETY

- Regularly check your ramp for worn, loose or damaged parts – If you find any of these to be damaged you must not use your ramp and you must contact your dealer immediately.
- Ensure ramp is fully extended and locked in position before use.
- Do not attempt to use ramps on gradients above 1 in 12. Please also check the gradient your wheelchair or scooter is capable of.
- Do not exceed the weight capacity of the ramp.
- Only allow one person/ mobility vehicle to use the ramp at once.
- The surface of the ramp may become slippery in icy or wet conditions. Please exercise caution in these conditions.
- Keep your ramp clean of any dirt or debris as this could interfere with its sliding motion.
- Always use the ramp in line with the wheelchairs/scooter manufacturer's guidelines; NEVER use it at a steeper gradient than recommended.
- Do not modify or add to your ramp as it could void any warranty claims.
- Do not use the ramp for anything other than its original intended purpose.
- Only clean your ramp with a mild detergent and ensure it is dry before use.

## **TECHNICAL INFORMATION**

Width 690mm  
Depth 610mm  
Height 790-900mm  
Net Weight: 6.9kgs

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

## **SERVICE WARRANTY**

Aidapt Bathrooms Ltd guarantees the product free from defects in material and workmanship for a period of one year.

Should this product be operated under conditions other than those recommended, or any attempts made to service or modify the product, then the warranty is rendered void.

The product you buy may differ slightly from illustrations. This warranty is in addition to, and does not affect your statutory rights.

Our guarantee is administered by our retailers.

If your product arrives damaged, you must contact the retailer from whom you bought it. The retailers contact details will be on the invoice that arrived with the product, or on the email you received when you placed the order. Do not contact Aidapt Bathrooms Ltd, only your retailer can arrange a replacement or refund.

If your product fails within the guarantee period, please contact the retailer from whom you bought it.

If you have received your product and require technical help, please call our help desk on 01744 745 000

A copy of the instructions leaflet can be downloaded from our website.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com)**  
**Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**